

**PPI®***The ultimate in precision & accuracy*Volume Viscosity **Device Type**

Auto Injector

Primary Container

Pre-Filled 2.25mL Glass Syringe

Dosage Type

Fixed | 10 µL

Usage

Multiple Dose (200 times) | Disposable

Needle Attachment

Manually Attached

Needle Insertion

Manual

Activation Method

Button Activated

Injection Feedback

Audible click at beginning of injection

Handling Instructions



1
Screw needle onto the device to attach



2
Twist the load knob clockwise until it comes to a stop



3
Hold device with the needle pointing up. Remove needle shield then press dose activator to prime



4
Insert needle to desired depth & press dose activator to inject. Move to next injection site & repeat



+
Rewind the device when delivery time exceeds 3 seconds

Key Features

- ✓ Accurate & precise injection
- ✓ Audible, visual & tactile feedback
- ✓ Ergonomic & innovative design
- ✓ Delivers small dosages
- ✓ One-handed operation
- ✓ Accommodates high viscosity formulations



The SHL Group logo and PPI are registered and/or non-registered trademarks of SHL Medical AG or its affiliates, in the United States, Europe and other jurisdictions. Contact info@shl-group.com for more information.

www.shl.group